



KEY BENEFITS & FEATURES



PRODUCT	RRP			
The Eye Doctor Dry Eye Kit	£22.50			
Includes:	Formula	USES	DETERGENT / PRESERVATIVE FREE	
	Essential Antibacterial Dry Eye Compress	N/A	200+	N/A
	Biodegradable Eyelid Wipes	0.9% Sodium Chloride	20	✓
	Intensive Relief Eye Drops	0.3% Sodium Hyaluronate	250	✓



Suitable for use with

The Eye Doctor Essential Compress, Biodegradable Eyelid Wipes and Intensive Relief Drops are suitable for Dry Eye Disease, MGD, Blepharitis, Chalazions and the associated symptoms, grittiness, irritation and sore eyes. Use The Eye Doctor Essential Compress cold for inflammation, headaches and Migraines.

When to dispense?

Patients suffering from common eye conditions should ideally incorporate all 3 products into their daily eyecare regime.

1. HEAT the eyelids to melt the oils causing blockages in the Meibomian Glands.
2. CLEANSE the eyelids and lashes by wiping away oil secretions and debris.
3. REVIVE the eyes with lubricating drops.

FAQ's

What makes The Eye Doctor Essential Eye Compress antibacterial?

Sterileyes® is an antibacterial shield permanently bound to the fabric fibres and clinically proven to reduce potentially harmful bacteria remaining on the eye compress surface by up to 99.9% after use. This patented, stay fresh, skin friendly technology inhibits the growth of odour causing bacteria over the compress surface. Sterileyes® does not interfere with your skin's natural bacterial flora.

The Sterileyes® fabric is antimicrobial to the three leading causative microorganisms of eye infections and that associated with dry eye development.

How often should my patient use the Dry Eye Kit?

The Eye Doctor Essential - use for 7-10 minutes, twice a day for two weeks. This can be reduced to once a day and then a couple times a week as required, once symptoms have been alleviated.

The Eye Doctor Biodegradable Eyelid Wipes – recommend one wipe per eye, twice a day as part of an eyelid hygiene regime. The Eye Doctor Intensive Relief Eye Drops – can be used as required, 4 times per day is recommended. If used 4 x a day your patient will have 31 days use.

How are The Eye Doctor Eyelid Wipes biodegradable?

Made from Viscose fibres, the wipes are fully biodegradable in natural environments: in the soil, compost and both fresh or marine water. They will fully convert back to nature and do not require recycling.

What is Sodium Hyaluronate in The Eye Doctor Intensive Relief Eye Drops?

Sodium Hyaluronate replenishes the tear film and effectively relieves dry, sore, gritty eyes. The Eye Doctor Intensive Relief Drops contains 0.3% Sodium Hyaluronate.

e sales@the-body-doctor.com

f +44(0) 1484868816

