

KEY BENEFITS & FEATURES



Product	RRP	Suitable for use with contact lenses?	Preservative Free?	Sterility	Classification
Screen Time	£10.00	Yes	Yes	6 months after opening	Class II A MD
Intensive Relief	£11.00	Yes	Yes	6 months after opening	Class II A MD
Intensive Mist	£12.50	Yes	Yes	6 months after opening	Class II A MD
The Eye Doctor® Advanced Triple Action Eye Drop	£15.00	Yes	Yes	6 months after opening	Class II A MD

FAQ's

How long do The Eye Doctor Drops/Mist last?

The Eye Doctor Screen Time and Intensive Relief Eye Drops can be used up to 250 times. The Eye Doctor Intensive Relief mist contains approximately 150 sprays, The Eye Doctor® Advanced Triple Action Drops can be used up to 300 times and all 4 products remain sterile for 6 months after opening.

How do The Eye Doctor Drops/Mist work?

The Eye Doctor Screen Time Eye Drops provide instant relief to sore eyes caused by over-exposure to digital screens. During prolonged screen use the blink rate dramatically decreases, this causes tears to evaporate and onset the dry, sore gritty eye symptoms.

The Eye Doctor Intensive Relief Eye Drops treat severe and persistent dry eye conditions. Relieves long-term dryness, irritation and burning caused by heating or air conditioning, smoke, dust, pollen and after surgery on the eye.

The Eye Doctor® Advanced Triple Action Eye Drops go beyond conventional Eye Drops. By addressing the three key factors contributing to dry eyes – strengthening the lipid layer, maintaining tear film stability, and preventing excessive dryness – this formulation offers a holistic approach to dry eye management.

The Eye Doctor Intensive Relief Eye Mist stabilises and protects the tear film and improves the moisturise of the eye surface and the eyelids. The mist helps improve tear film stability that is affected by external and environmental factors.

Which products can be used with contact lenses?


All eye drops and the mist can be used with hard or soft contact lenses. Your patient does not need to remove their contact lenses before application.









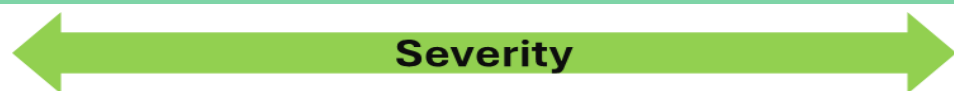

How should my patient use the product?

All drops can be used as required, 4 times per day is recommended. The Eye Doctor Intensive Relief Mist can be used as required, we recommend spraying once or twice, 3 – 4 times per day.



Which Eye Drop or Mist is right for my patient?



	 Screen Time Eye Drops	 Intensive Relief Eye Drops	 Intensive Relief Eye Mists	 Triple Action Eye Drops
 Ingredients	0.1% Sodium Hyaluronate	0.3% Sodium Hyaluronate	Liposomes & Pro-Vitamin B5	0.1% sacha inchi (Plukenetia volubilis) seed oil, 2% trehalose, 0.2% high-molecular-weight hyaluronic acid, 0.9% glycerol.
 Bottle Size	10ml	10nl	10nl	10ml
 Number of Applications	250	250	130	300
 Discomfort Level	Mild to moderate dry eyes	Severe, persistent dry eyes	Severe, persistent dry eyes	Severe, persistent dry eyes
 Application Mechanism	Easy to use dispensing pump	Easy to use dispensing pump	Spray	Easy to use dropper bottle
 Indication	<ul style="list-style-type: none"> Screen users Frequent contact lens wearer First line soothing relief 	<ul style="list-style-type: none"> Post eye surgery relief Long term dry eye sufferers Suffers of burning, itching red eyes 	<ul style="list-style-type: none"> Irritated dry eye caused by environmental factors Soothing to skin around the eyes Makeup wearers 	<ul style="list-style-type: none"> Chronic Dry Eyes Severe Dry Eye Conditions Suitable for preventive care Daily Eye Care Routine

E - sales@the-body-doctor.com
 T - +44(0) 1484868816





Reminders & Precautions

The Eye Doctor Eye Drops

- Advise your patient vision may be temporarily affected immediately following the use of the product.
- Advise your patient not to use simultaneously with prescribed eye medicines.

The Eye Doctor Intensive Relief Mist

- Your patient should not use The Eye Doctor Intensive Relief Mist if they are hypersensitive to any of the ingredients.
- If your patient is using additional eye drops/eye ointments, they should leave a 10-minute time interval between the applications.
- The Eye Doctor Intensive Relief Mist is make-up resistant; it can be used

Your patient should not use any of the products if they are hypersensitive or allergic to any of its ingredients. Carefully review the ingredient list provided on the packaging to ensure there are no known allergens that may cause an adverse reaction.

If your patient experiences any signs of an allergic reaction, such as itching, redness, swelling, or discomfort, please advise to discontinue use immediately and consult with a healthcare professional.

Store the eye drops in a cool, dry place, away from direct sunlight and extreme temperatures. Keep the product out of the reach of children.