












KEY BENEFITS & FEATURES



Product	RRP	Suitable for use with contact lenses?	Preservative Free?	Sterility	Classification
Screen Time	£10.00	Yes	Yes	6 months after opening	Class II A MD
Intensive Relief	£11.00	Yes	Yes	6 months after opening	Class II A MD
Intensive Mist	£12.50	Yes	Yes	6 months after opening	Class II A MD

Which Eye Drop or Mist is right for my patient?

Severity	Moderate	Severe	
	Screen Time Eye Drops 	Intensive Relief Eye Drops 	Intensive Relief Eye Mist 
 Ingredients	0.1% Sodium Hyaluronate	0.3% Sodium Hyaluronate	Liposomes & Pro-Vitamin B5
 Bottle Size	10ml	10ml	10ml
 Number of applications	250	250	130
 Discomfort level	Mild to moderate dry eyes	Severe, persistent dry eyes	Severe, persistent dry eyes
 Application mechanism	Easy-to-use dispensing pump	Easy-to-use dispensing pump	Spray
 Indication	<ul style="list-style-type: none"> - Screen users - Frequent contact lens wearer - First line soothing relief 	<ul style="list-style-type: none"> - Post eye surgery relief - Long term dry eye sufferers - Sufferers of burning, itchy red eyes 	<ul style="list-style-type: none"> - Irritated dry eyes caused by environmental factors - Soothing to the skin around the eyes - Make-up wearers

FAQ's

How long do The Eye Doctor Drops/Mist last?

The Eye Doctor Screen Time and Intensive Relief Eye Drops can be used up to 250 times. The Eye Doctor Intensive Relief mist contains approximately 150 sprays, and all 3 products remain sterile for 6 months after opening.

How do The Eye Doctor Drops/Mist work?

The Eye Doctor Screen Time Eye Drops provide instant relief to sore eyes caused by over-exposure to digital screens. During prolonged screen use the blink rate dramatically decreases, this causes tears to evaporate and onset the dry, sore gritty eye symptoms.

The Eye Doctor Intensive Relief Eye Drops treat severe and persistent dry eye conditions. Relieves long-term dryness, irritation and burning caused by heating or air conditioning, smoke, dust, pollen and after surgery on the eye.

Both Eye Drops contain a natural occurring substance called Sodium Hyaluronate, which replenishes the tear film and effectively relieves dry, sore, gritty eyes. The Eye Doctor Screen Time Drop contains 0.1% Sodium Hyaluronate, and The Eye Doctor Intensive Relief Mist contains 0.3% Sodium Hyaluronate.

The Eye Doctor Intensive Relief Eye Mist stabilises and protects the tear film and improves the moisturise of the eye surface and the eyelids. The mist helps improve tear film stability that is affected by external and environmental factors.

Which products can be used with contact lenses?

Both eye drops and the mist can be used with hard or soft contact lenses. Your patient does not need to remove their contact lenses before application.

How should my patient use the product?

The Eye Doctor Screen Time and Intensive Relief Eye Drops can be used as required, 4 times per day is recommended. The Eye Doctor Intensive Relief Mist can be used as required, we recommend spraying once or twice, 3 – 4 times per day.

If used 4 x a day your patient will have 31 days use of the The Eye Doctor Screen Time and Intensive Relief drops and 16 days use of the The Eye Doctor Intensive Relief Mist.

Reminders & Precautions

The Eye Doctor Screen Time & Intensive Relief

- Advise your patient vision may be temporarily affected immediately following the use of the product.
- Advise your patient not to use simultaneously with prescribed eye medicines.

The Eye Doctor Intensive Relief Mist

- Your patient should not use The Eye Doctor Intensive Relief Mist if they are hypersensitive to any of the ingredients.
- If your patient is using additional eye drops/eye ointments, they should leave a 10-minute time interval between the applications.
- If discomfort persists or is getting worse, tell your patient to contact you for further advise.
- The Eye Doctor Intensive Relief Mist is make-up resistant; it can be used on your patient's eye lids over make up is applied which will not run or smudge your patients make up.

e sales@the-body-doctor.com

f +44(0) 1484868816

