

WARNINGS & CAUTIONS

Use NuLids only under care of Optician/Optomtrist

Discontinue treatment & consult your eye care professional if discomfort on eyelids or eyeball is noted.

Always use the lubricant gel or lid hygiene product recommended by your Optician/Optomtrist

DO NOT re-use Soft Tips. Dispose after a single treatment to avoid infection or injury.

DO NOT share Soft Tips with another person.

KEEP the handpiece dry and free from excess liquids.

DO NOT put the Soft Tip on your eye or anything other than the eyelids.

DO NOT conduct treatment more frequently or for a longer period than recommended.

AVAILABLE RESOURCES

Visit www.the-body-doctor.com to Download eBrochure

- Watch video on how to use NuLids
- Find contact information and our other eye care ranges

THE BODY DOCTOR

Email: customerservices@the-body-doctor.com

Telephone: 01484 868816

WE'RE PERFECT TOGETHER

The Ultimate Dry Eye Package for at Home Treatment



Eye Compress...Hot & Cold eye compress for the treatment Dry Eye Disease and the associated symptoms of Dry, Sore and Gritty Eyes



NuLids...An all-natural, one minute per day at-home treatment for dry eye disease that really works! FDA Registered!

Exclusively from



Head Office:
Unit 7, Denby Dale Industrial Estate,
Wakefield Road, Denby Dale,
Huddersfield. HD8 8QH.
T: +44 (0)1484 868 816
W. www.the-body-doctor.com

NuLids®

QUICK START GUIDE*

