

# 5 STEPS TO FLIP THE UPPER LID WITH THE MEIVERTOR™



With the patient looking down, position the lashes between the tips of the Meivertor with the large tip at the top.



Grasp the lashes firmly, being sure not to grab the lid.



Pull the upper-lid slightly down and away.



Keeping the pads firmly closed around the lashes, rotate your thumb up and away from the patient



Continue to rotate until the inside of the upper lid is exposed, and the lid is fully everted.



## THE LOWER LID



The lower lid is easy, but you knew that already. Just press the side of the smaller tip into the lower lid margin, pull down, and roll forward slightly.

T: +44 (0) 1484 868 816  
E: sales@the-body-doctor.com  
W: www.the-body-doctor.com



**The ONLY  
Single-Handed  
Upper and Lower  
Eyelid Eversion Tool**



\*UK Patent pending

